Bumpi® Rings

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Produkt d'hygiène
Made in Germany

Bumpi® Rings

Protects her, helps him – to love stress-free.

Indications:
Women with endometriosis, prolapse of the uterus or myomas sometimes experience bleeding or pain during sex. The medical name for this problem is collision dyspareunia.

Despite a variety of causes, there is a simple solution: the soft Bumpi® rings are slipped over the penis and in this way prevent the partner penetrating too deep. The result: pain-free sex, no more anxiety about injuries and finally relaxed, fulfilling sexuality.

Note:
Even men with a very large penis can use the Bumpi® rings comfortably (approx. 6 % of men are affected by opulent male genital syndrome, OMG in short).

Application:
One or several soft Bumpi® rings are stretched with both hands, and, before penetration, pulled up the penis shaft right up to the testicles. The depth of penetration can be adjusted individually, depending on the number of Bumpi® rings used. To pull off, the rings are again stretched and can then be removed.

Using the Bumpi® ring is simple, quick and does not interfere with lovemaking. There is no risk of the Bumpi® rings slipping inside the partner.

Cleaning:

The soft Bumpi® rings should be cleaned with water and soap before the first use and after each use. After drying, they can be stored securely in the supplied case.

Material:
TPE (thermoplastischer elastomer)

Advantages:

- friendly to the skin, soft, flexible
- compatible with water-soluble lubricants, silicone oil, skin creams, vegetable fats
- biocompatible, tested to ISO 10993
- compatible with condoms made from latex and polyisoprene
- free from latex, phthalate, bisphenol A and silicone
- biocompatible, gem. ISO 10993 geprüft
- durable and reusable

Note:
Only suitable for external use. Do not use the product if there is known intolerance to its material.

Contents:
• 3 Bumpi® rings
• storage case
• user instructions
• 2 x sachets Sylk lubricant

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